

Directions for Use

Tray Adhesive for Silicones

Tray adhesives are recommended to assure reliable adhesion of impression materials to the impression tray and to avoid uncontrollable deformation during impression taking.

Oxford Tray FIX secures safe adhesion between addition curing or condensation curing silicones and metal - or methacrylate trays.

Note:

The adhesive does not always secure sufficient adhesion to trays made of other plastic materials or individual trays made of base plates.

A polyether adhesive must be used for polyether impression materials.

Contraindications

Contraindications or interactions are not known.

Working Instructions

Apply a thin layer of Oxford Tray FIX to the tray using the applicator brush and allow to dry at least **3 minutes**. Then load the tray and take the impression.

Close bottle immediately after use!

Cleaning from the tray

Oxford Tray FIX can be removed from the tray with acetone.

Warnings

- Oxford Tray FIX should only be used in well ventilated rooms.
- Keep away from sources of ignition.
- In case of contact with skin, wash off with water.

Storage

Do not store above 25 °C (77 °F)! Do not use after expiration date.

Warranty

First Scientific Dental Materials GmbH warrants this product will be free from defects in material and manufacture. First Scientific Dental Materials makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and First Scientific Dental Materials' sole obligation shall be repair or replacement of the First Scientific Dental Materials product.

Limitation of Liability

Except where prohibited by law, First Scientific Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!

For dental use only!

Caution:

Federal law restricts the sale of this device to or by the order of a dentist.



Manufacturer:

First Scientific Dental Materials GmbH, 25335 Elmshorn, Germany

